REF	1541	1571
	1.2 mm (0.049 in.)	1.2 mm (0.049 in.)
	10cm (4 in.)	10cm (4 in.)
Needle Aiguille Ago Aguja Nadel	18Ga	18Ga
Units Unités Unità Unidades Einheiten	 2 x 8 = 16	

OCCCLUDING and RETRACTING LOOP

DESCRIPTION:
Radiopaque silicone tubing, silicone adhesive, stainless steel needle and thermoplastic clamp.

INDICATIONS FOR USE:
Radiopaque silicone tubing with sealed ends to provide an air-cushioned surgical tool for manipulating anatomical organs and structures to either retract them or ligate vessels with minimum trauma.

WARNING:
Intended for single patient use only. DO NOT REUSE.
After use, this product may be a potential biohazard. Handle and discard in accordance with acceptable medical practice and applicable local, state, and federal laws and regulations.

CAUTION:
Prior to use, examine the package carefully before opening to confirm its integrity and the expiration date has not passed. The device is supplied in a sterile package and is non-pyrogenic. Do Not Use if package is damaged, opened or the expiration date has passed.

Sterilized by Ethylene Oxide. **Do Not Resterilize.** RX only.

CONTRAINDICATIONS: None known.

- INSTRUCTIONS FOR USE:**
- Ensure distance 'D' is greater than length 'L'
 - CAUTION:** Do not use excessive force when pushing needle through locking mechanism.
 - When using to occlude, slightly stretch tubing to tighten before sliding into slot of keyhole locking mechanism.

For symbols glossary visit www.questmedical.com/resources.aspx

MDI **STERILE EO** **EC REP**

QuestMedical.com/eIFU

QUEST Medical, Inc.
Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226
F +1.972.390.8465

Printed in USA
903596 REV D
FEB 23

© 2019 Quest Medical, Inc. All rights reserved.
Retract-O-Tape and SaddleLoop are registered trademarks of Quest Medical, Inc.

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