Retract-O-Tape®

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Instructions for use (without needle): The tape is stretched and each end is secured with sufficient tension to occlude vessel.

U.S. Patent 4.140.125

For symbols glossary visit www.questmedical.com/resources.aspx

One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226
F +1.972.390.8465

Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

Printed in USA
904580 REV C
FEB 23

© 2023 Quest Medical, Inc. All rights reserved.
Retract-O-Tape is a registered trademark of Quest Medical, Inc.

Retract-O-Tape®

OCCCLUDING and RETRACTING LOOP

DESCRIPTION:

Radiopaque, round, hollow, sealed silicone tubing with or without stainless steel needle.

INDICATIONS FOR USE:

Radiopaque silicone tubing with sealed ends to provide an air-cushioned surgical tool for manipulating anatomical organs and structures to either retract or ligate vessels with minimum trauma.

WARNING:

Intended for single patient use only. DO NOT REUSE.
After use, this product may be a potential biohazard. Handle and discard in accordance with acceptable medical practice and applicable local, state and federal laws and regulations.

CAUTION:

Prior to use, examine the package carefully before opening to confirm its integrity and the expiration date has not passed. The device is supplied in a sterile package and is non-pyrogenic. Do Not Use if package is damaged, opened or the expiration date has passed.

Sterilized by Ethylene Oxide. **Do Not Resterilize.** RX only.

CONTRAINDICATIONS: None known.

DIRECTIONS FOR USE:

Retract-O-Tape® with Needle

1041 - 12" RETRACT-O-TAPE
Vascular Loop with Blunt Needle, 18 gauge (0.029" x 0.049")

1071 - 12" RETRACT-O-TAPE
Vascular Loop with Semi-pointed Needle, 18 gauge (0.029" x 0.049")

Instructions for use (with needle): Begin by piercing the myocardium about two or three millimeters to the side of the coronary artery, passing the needle tape under and around the artery and exiting the myocardial tissue two or three millimeters on the other side of the artery.

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